

“HEMODIALYSIS MACHINE DISINFECTANTS”



6009 SURFACTO™ NEUTRAL

Hemodialysis Machine Surface Cleaner Disinfectant

Hemodialysis machines surface cleaning and disinfection between treatments. Suitable for efficient cleansing of dialysis machine surfaces.

Ingredients

Alkyl dimethyl benzyl ammonium chloride (0,1%), dodecyl trimethyl ammonium bromide, propylene glycol, benzotriazole, ethanol 96%, stabilizing agent, and RO water. Free of glutaraldehyde, formaldehyde, buffers, and other phenol derivatives.

Features

- ☑ Detergent disinfectant.
- ☑ Free of alcohol, aldehyde and phenol.
- ☑ Ready-to-use.
- ☑ Active from 1 minute.
- ☑ Wide material compatibility; especially with acrylic, glass and PVC.
- ☑ Quick drying: does not leave trace or greasy film.



Microbiological Properties

Effect	Standard	Microorganisms (with ATCC)	Contact Time	Concentration of use
Bactericidal	EN 13727	Staphylococcus aureus 6538 Pseudomonas aeruginosa 15442 Enterococcus hirae 10541	1 minute	Ready-to-use
Fungicidal	EN 13624	Candida albicans 10231	5 minutes	Ready-to-use
		Aspergillus brasiliensis 16404	60 minutes	Ready-to-use

Instruction

Ready-to-use. Spray on the medical device surface. Wait until the desired duration of action is indicated in the table. Wipe with a sterile wipe.

Packaging

1 carton: 1 L sprayer bottle x 20

1 pallet: 42 cartons / 840 bottles

Order No: 436009051000

1 carton: 5 L jerrycan with 38 mm cap x2

1 pallet: 54 cartons / 108 cans

Order No: 436009015000

Purity in Every Drop
Effective, Quality and Sustainable Products

GBL GÜL BİYOLOJİ LABORATUVARI SANAYİ VE TİCARET ANONİM ŞİRKETİ

Şerifali Mahallesi Hattat Sokak No:10 34775 Ümraniye İstanbul TÜRKİYE

Tel: +90 216 364 15 00 - Fax: +90 216 314 15 69

www.gbl.com.tr - export@gbl.com.tr

www.dezenfeksiyon.com.tr